



ASP
SCOPEVALET™
ENDO BAG & TAG

Simple. Secure. Compliant.


The **SCOPEVALET™ ENDO BAG & TAG** is an innovative system designed to transfer clean and contaminated endoscopes to and from the procedure room. Featuring color coded, uniquely numbered, tamperproof tags, the **SCOPEVALET™ ENDO BAG & TAG** offers a safe, protective environment and clear identification of the enclosed endoscope. For the newly disinfected scope, the green tag indicates that the scope is "clean". After the procedure, the scope is returned to the **SCOPEVALET™ ENDO BAG & TAG** bag and the red tag is attached, clearly showing the enclosed equipment is "dirty". The scope can then be safely transported back to the cleaning room in its sealed system, easily identified as contaminated and ready for reprocessing.

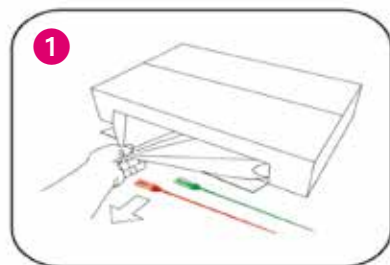


Open the front of the box and remove one bag and one each of the GREEN and RED tags.

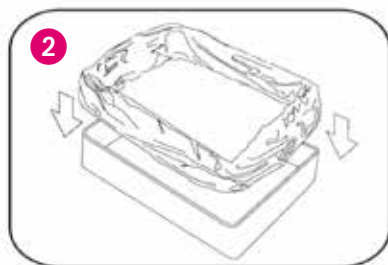


Place the bag on an appropriate surface ready for transportation.

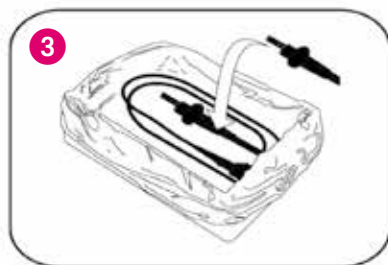


Place the reprocessed endoscope into the bag.

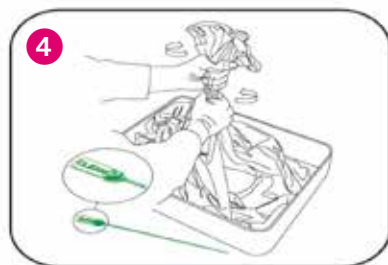

Bring the bag edges together and press down to remove the air. Twist the top of the bag.

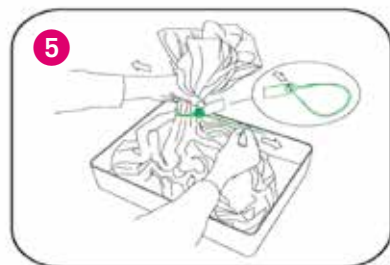


Place the GREEN tag around the bag and lock into position. Take the scope to the procedure room along with the RED tag.

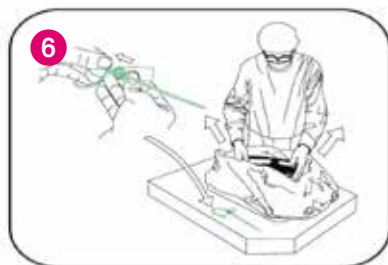

Break the GREEN tag open and discard.




Undertake the procedure.

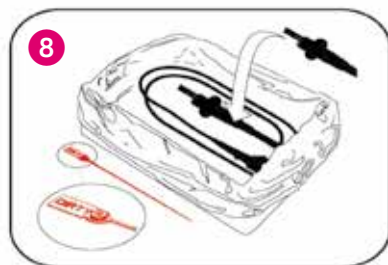


Place used scope into the bag after bedside cleaning.




Bring the bag edges together and press down to remove the air. Twist the top of the bag.

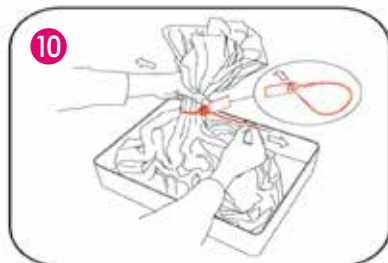

Place the RED tag around the bag and lock into position. Take scope to the cleaning room.

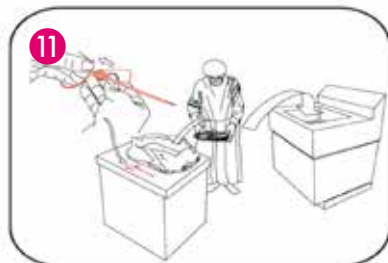

Break the RED tag open and remove the scope.

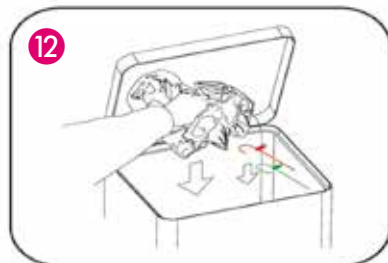

Discard the bag and RED tag.

The **SCOPEVALET™ ENDO BAG & TAG** is distributed by Advanced Sterilization Products
 EC Rep.: Ruhof UK, Unit 4-5 Brampton Business Park
 55 Brampton Road, Eastbourne East Sussex BN22 9AF
SCOPEVALET™ is a trademark of Ruhof Corporation.
 ©Ethicon US, LLC 2014. All rights reserved.

ADVANCED STERILIZATION PRODUCTS

Division of Cilag GmbH International
 a **Johnson & Johnson** company

ORDERING INFORMATION:

PRODUCT CODE	DESCRIPTION	CONTENTS
9102905	SCOPEVALET™ ENDO BAG & TAG	100 Endo Bags 100 Green Tags 100 Red Tags