



# CIDEZYME®

ENZYMATIC DETERGENT SOLUTION

## MANUAL CLEANING

CIDEZYME® Enzymatic Detergent Solution is a mild enzyme-based pre-soak and cleaner designed to effectively remove organic soil from contaminated endoscopic instruments and other medical devices prior to disinfection.

## EXPERIENCE THE BENEFITS

CIDEZYME® Enzymatic Detergent Solution is recommended to clean medical instruments before using the ASP High-level disinfection solutions.

- **Effective and thorough action with protease enzyme and surfactants**
- **Fast-acting**
- **User-friendly**
- **High compatibility with materials**





# CIDEZYME®

ENZYMATIC DETERGENT SOLUTION

## FEATURES & BENEFITS

- **EFFECTIVE WITH PROTEASE ENZYME** that breaks down proteins like blood, mucous and feces contained in the biofilm with reduced mechanical action.
- **EFFECTIVE WITH SURFACTANTS** that improve the surface wetting of hard-to-reach areas like endoscope channels, and effectively clean away lipids and greases.
- **FAST-ACTING** – begins to act in just one minute allowing fast turnaround of instruments.
- **EASY RINSING** – its low foaming properties and neutral pH prevent it from leaving residues so that instruments can be effectively disinfected.
- **PLEASANT TO USE** with spearmint fragrance that masks the odors of dirty instrumentation.
- **EASY DISPENSING** with separate dosing pumps available.

## MATERIALS COMPATIBILITY

CIDEZYME® Enzymatic Detergent Solution can be used to clean a wide range of devices made from metals, plastics and elastomers, making it a very versatile cleaner that can be used anywhere medical instruments need to be thoroughly cleaned.

## TECHNICAL INFORMATION

MEDICAL DEVICE CLASSIFICATION	Class I according to MDD 93/42/EEC
IN-USE CONCENTRATION	Dispense 8 ml (0.8%) or 16 ml (1.6%) per liter of water
SOAK TIME	1 to 3 minutes
USE LIFE	Use fresh as required
SHELF LIFE	2 years
STORAGE	15-30°C
DISPOSAL	Drain or as per hospital policy

## ORDERING INFORMATION

REORDER NO.	DESCRIPTION	CASE CONTENTS
2258	CIDEZYME® Enzymatic Detergent Solution	12 x 1 liters
2260	CIDEZYME® Enzymatic Detergent Solution	4 x 5 liters



CIDEZYME® Enzymatic Detergent Solution may also be used for cleaning instruments in automated equipment, in accordance with equipment manufacturers' recommendations.

Please read and follow the Instructions for Use prior to using CIDEZYME® Enzymatic Detergent Solution for detailed information, including contraindications, warnings and proper directions for use.

**FOR MORE INFORMATION, CONTACT YOUR LOCAL ADVANCED STERILIZATION PRODUCTS SALES REPRESENTATIVE**

[www.aspjj.com/emea](http://www.aspjj.com/emea)

**UK:**  
Advanced Sterilization Products  
A Division of J&J Medical Ltd.  
Pinewood Campus  
Nine Mile Ride  
Wokingham  
Berkshire RG40, 3EW, England  
T: +44 1344 871 081  
F: +44 1344 871 171

**Ireland:**  
Advanced Sterilization Products  
A Division of J&J Medical Ireland  
Airton Road, Tallaght,  
Dublin 24, Ireland  
T: +353 1 466 5200  
F: +353 1 466 5340

**Egypt:**  
Johnson & Johnson Medical Egypt  
Florida Mall  
5th Floor  
1229 Square EL Sheikh Ali Gad EL  
Hak St. Heliopolis  
Cairo, Egypt  
T: +202 2268 5026  
F: +202 2268 4674

**Middle East:**  
Advanced Sterilization Products  
A Division of J&J Medical Middle East  
Dubai Healthcare City, J&J building,  
3rd floor  
PO Box 505080, Dubai,  
United Arab Emirates  
T: +971 4 429 7235  
F: +971 4 429 7250

**South Africa:**  
Advanced Sterilization Products  
A Division of J&J Medical Pty.  
Ltd. SA  
PO Box 273, Midrand  
Halfway House  
1685  
South Africa  
T: +27 11 265 1120  
F: +27 11 265 1189

## ADVANCED STERILIZATION PRODUCTS

Division of Medos International Sàrl

a **Johnson & Johnson** company