CIDEX®

Instrument Tray System

REF 82010 / REF 82016 / REF 82027

REF 82032 / REF 82076

US

GB

IE

CA

INSTRUCTIONS FOR USE

Refer to the manufacturers instructions for use and precautions for soaking, cleaning or disinfecting solutions.

CLEANING

Before using the tray for the first time, or when changing solutions, thoroughly wash (using detergent) and rinse the tray. Always follow the manufacturers instructions for the detergent.

- The tray can either be air dried, or dried with a lint free cloth.
- When the tray is completely dry, new solution may be poured into the tray.

STERRAD® STERILIZATION SYSTEM

- To allow for proper processing, wrap the tray, lid and inner tray (if supplied) separately.
- Follow STERRAD® Systems Operators Guide for wrapping and proper loading instructions.

STEAM STERILIZATION

- To allow for proper processing, wrap the tray and inner tray (if supplied) together. Place a towel inside the tray. Wrap the lid separately.
- CIDEX® Trays can be processed using a gravity, pre-vacuum or flash steam cycle. CIDEX Trays can be exposed to temperatures up to 134°C (273°F) for 4 minutes.
- Always follow the manufacturers instructions or facility guidelines for operation of steam sterilizer.

WASHER DECONTAMINATOR

- To allow for proper decontamination, process the tray, lid and inner tray (if supplied) separately.
- CIDEX Trays can be exposed to temperatures up to 134°C (273°F).
- Always follow the manufacturers instructions or facility guidelines for operation and loading of the washer/decontaminator.

Reusable labels for identifying the contents of the tray when in use are provided for your convenience.

ADVANCED STERILIZATION PRODUCTS

Division of Ethicon, Inc.

a company

33 Technology Drive, Irvine, CA 92618-9824

For technical information please contact your ASP representative

	Johnson & Johnson Medical GmbH Oststraße 1, 22844 Norderstedt, Germany
	Irvine, CA 92618
	92787 Issy-les-Moulineaux
	22844 Norderstedt
	8957 Spreitenbach
	3800 AD Amersfoort
	15125 Marousi, Athens
	19184 Sollentuna
	1700 Dilbeek
	Johnson & Johnson, s.r.o. Karlův English 3201/6 150 00 Praha 5
	Plynárenská 7/B 824 78 Bratislava 26
	Johnson & Johnson Medical S.p.A. Via del Mare, 56 - 00040 Pomezia, Roma
	28042 Campo de las Naciones, Madrid
	Ascot, SL5 9EY
	Johnson & Johnson Medical Products Markham, ON, L3R 0T5
	ジョンソン・エンド・ジョンソン 株式会社 〒101-0065 東京都千代田区西神田 3丁目5番2号
	1190 Wien
	Rodovia Presidente Dutra, Km 154 S.J. Campos-S.P. 12240-908
	2745-555 Barcarena
	Źłocka 24, 02-135 Warszawa
	Johnson & Johnson Kft. H-2045 Törökbalint, Tó Park

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© Ethicon, Inc. 2006-2009