



Competency Checklist: STERRAD® CYCLESURE® 24 Biological Indicator

Name of Employee : _____ Name of Facility: _____

COMPETENCY	REFERENCE	RETURN DEMO	EVALUATOR VERIFY INITIALS
Identify indications for use of the STERRAD® CYCLESURE® 24 Biological Indicator.	AD-51868-001 CO-1002495		
Demonstrate set-up. <ul style="list-style-type: none"> •Check expiration date on box •Ensure integrity of vial has not been compromised •Check media color •Label unprocessed vial •Place vial in proper sized pouch 	AD-51868-001 CO-1002495		
Prepare STERRAD® CYCLESURE® 24 Biological Indicator for each sterilization cycle.	AD-51868-001 CO-1002495		
Discuss and demonstrate proper placement of vial in sterilizer. <ul style="list-style-type: none"> • placement in chamber 	AD-51868-001 CO-1002495		
Demonstrate steps after processing. <ul style="list-style-type: none"> •gloves to be worn •check chemical indicator for correct color change •correct seating of the cap •correct crushing of the vial •correct position of vial after crushing 	AD-51868-001 CO-1002495		
Demonstrate incubation. <ul style="list-style-type: none"> •note temperature of the incubator •document temperature 	AD-51868-001 CO-1002495 AD-110402-01-US AD-110401-01-US		
Discuss process and use of controls.	AD-51868-001 CO-1002495		
Discuss reading the results. <ul style="list-style-type: none"> •color changes •24 hour reading •72 hour disposal •documentation 	AD-51868-001 CO-1002495		
Discuss hospital/facility policy on handling positive BI.	AD-51868-001		
Discuss ASP recommendation on handling positive BI.	AD-51868-001 CO-1002495		
Discuss disposal of STERRAD® CYCLESURE® 24 Biological Indicator.	AD-51868-001 CO-1002495		
Discuss storage of STERRAD® CYCLESURE® 24 Biological Indicator.	AD-51868-001 CO-1002495		

Employee signature: _____
 Evaluator: _____
 Date: _____